

Biopolymeric Hydrogel Surgical Dressing

ReliHeal[®]-G

Product Description

ReliHeal[®]-G is a single use sterile gauze supported hydrogel wound dressing. The formulation provides a moist wound environment that enhances wound healing and helps prevent wound desiccation. ReliHeal[®]-G absorbent properties allow it to be used on exuding wounds.

Ingredients:

Polyvinyl alcohol: 3.5 G
Polyethylene glycol: 0.09 G
De-ionized water: q.s.

Indications

ReliHeal[®]-G dressing is intended to be used as a primary dressing on partial and full-thickness wounds, including diabetic ulcers, venous stasis ulcers, pressure ulcers, surgical wounds, ischemic ulcers, traumatic wounds, superficial burns and donor sites.

Contraindications

ReliHeal[®]-G dressing is not indicated for use on:
Third-degree burns (post debridement)
Highly oozing and infected wounds

Precautions

ReliHeal[®]-G dressing may be used when visible signs of infection are present in the wound area only when proper medical treatment addresses the underlying cause.
Do not use on patients with known sensitivity to polyvinyl alcohol and polyethyleneglycol.
ReliHeal[®]-G dressing is not intended to control bleeding.

Direction for the Use:

1. Clean wound and its surrounding skin. In case of excessive bleeding, control the bleeding before application of the dressing.
2. Tear the pouch and remove the tray from the pouch.
3. Remove the transparent cover on the hydrogel.
4. Hold the gauze and gently peel the ReliHeal[®]-G hydrogel from the tray.
5. Apply on the wound bed and secure with a secondary dressing like gauze.

NB: In case of requirement of any anti-infective cream on the wound, apply the cream first followed by ReliHeal[®]-G dressing as outlined above.

Removing the Dressing:

ReliHeal[®]-G dressing can be left undisturbed on the wound for 72 hrs. However, depending on the nature of the wound and volume of drainage, the dressing might require a higher frequency of change. It is recommended that the dressing is monitored 48 hrs after its application. Change the dressing as required.

Dressing Change:

1. ReliHeal[®]-G dressing is recommended to be replaced 72 hrs after its application in low-moderate exuding wounds.
2. ReliHeal[®]-G dressing is to be examined 48 hrs after its application. Appearance of a translucent /opaque hydrogel indicates the need for dressing change. If dressing change is required, gently remove the ReliHeal[®]-G dressing, otherwise cover ReliHeal[®]-G dressing with a fresh secondary dressing.
3. At the time of dressing change, rinse away remaining gel with wound cleanser or sterile normal saline as per practice.
4. At the time of dressing change, if the dressing is adhered to the wound surface, saturate with wound cleanser or sterile normal saline, allow the dressing to soften, and gently remove.
5. Avoid forceful removal of the dressing to minimize disruption of the wound bed.

Storage and Shelf Life:

For best results, store in cool and dry place at room temperature.
Avoid excessive heat and humidity. In case of excessive heat (above 40°C) it is recommended to be stored in the refrigerator. Shelf life is 2 years from the date of manufacture.

How Supplied:

Each box contains 3 individual pouches with dressing. Sterility of product is guaranteed unless the individual pouch is damaged or open.

How to use:

